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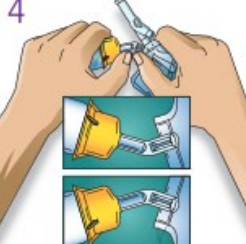
[www.landmarkinfusion.com](http://www.landmarkinfusion.com)

**MINI-BAG PLUS system directions:**  
**Only for Single Dose Powdered Drug Vials with 20 mm Closures**  
**Use Aseptic Technique**

<p><b>1</b></p> 	<p><b>2</b></p> 	<p><b>Assembly</b></p> <ol style="list-style-type: none"> <li>① • Remove vial cover</li> <li>• Disinfect stopper</li> <li>② • Peel off foil cover</li> <li>• Inspect adaptor for moisture</li> <li>Discard if found</li> <li>③ • Place vial upright</li> <li>• Hold firmly</li> <li>• Push adaptor down until vial snaps in place</li> <li>• DO NOT TWIST</li> <li>• Pull vial to ensure fully seated</li> </ol>
<p><b>3</b></p> 	<p><b>4</b></p> 	<p><b>Reconstitution</b></p> <ol style="list-style-type: none"> <li>④ • Squeeze bag and check vial</li> <li>• Use only if vial fully seated and dry</li> <li>• Bend up then down to break seal</li> <li>⑤ • Hold bag with vial down</li> <li>• Squeeze solution into vial until <b>half full</b></li> <li>• Shake to suspend drug in solution</li> <li>⑥ • Hold bag with vial upside down</li> <li>• Squeeze bag to force air into vial</li> <li>• Release to drain suspended drug from vial</li> <li>• Repeat steps 5 and 6 until vial is <b>empty</b> of drug and solution is thoroughly mixed. <b>Ensure drug is completely dissolved. Do Not Remove Drug Vial.</b></li> </ol>
<p><b>5</b></p> 	<p><b>6</b></p> 	<ol style="list-style-type: none"> <li>⑦ • Remove port protector.</li> <li>• Attach administration set per its directions.</li> <li>⑧ • Hang container on I.V. pole and prime set per directions. Ensure that vial is empty of drug and solution. Repeat step 6 if drug and solution remain in vial.</li> <li><b>Warning:</b> Do not use in series connections</li> <li>⑨ • Administer medication per directions. Use within specified time for drug stability. Refer to drug package insert.</li> </ol>

Rx only.  
 To ensure safe and proper use of this device, refer to the complete instructions for use.

To Order Baxter Products, call 1-888-229-0001

Baxter, Mini-Bag Plus and Committed to a Safer Healthcare Environment are trademarks of Baxter International Inc.  
 Baxter Healthcare Corporation, Medication Delivery, Route 120 and Wilson Road, Round Lake, IL 60073 www.baxter.com 111094 5K 04/09

Options at work.

MEDICATION DELIVERY

**FIGURA 1**  
 \*ver pasos 4-6

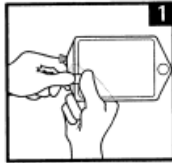
Irving: 8200 Tristar Drive, Suite 120  
 Texas 75063

Irving,

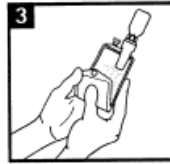
Fort Worth: 3515 NW Jim Wright Freeway  
 Fort Worth, Texas 76106

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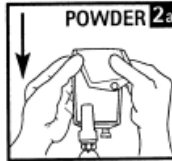
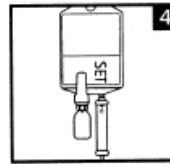
**1** At time of use, check to ensure addEASE is firmly latched to container. If not firmly latched, discard assembly. Prior to reconstitution, inspect contents of drug vial for presence of liquid. If liquid is present in powdered drug, discard assembly.

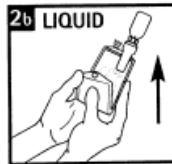
With vial positioned above container, as shown, fold container a third from top and squeeze firmly to transfer solution from vial to container. If necessary, repeat Steps 2 and 3. **CAUTION:** Inspect solution and discard assembled system if cloudiness, haze, or particulate matter is found.

To activate vial containing *powder drug*, position vial below container **2a**. For vial containing *liquid drug*, position vial above container to activate **2b**. Fold container a third from top and squeeze firmly to force plug from spike into vial. To transfer diluent to vial, fold container a third from top and squeeze firmly. Repeat until vial is approximately half full. Do not overfill vial with liquid. Thoroughly mix to reconstitute, following drug manufacturer's Package Insert.

After reconstitution, attach the administration set to container set port. Refer to IV set Instructions for Use. Hang container. To prevent fluid from re-entering vial, avoid squeezing or flexing container. **CAUTION:** Do not disconnect medication vial or PAB container from addEASE. After use, dispose of assembled system per institutional protocol.

Fluid path is:

**STERILE R**  
**NONPYROGENIC**

Not made with natural rubber latex or DEHP.

Compatible with most 20 mm medication vials.

 For single use only.  
 Do not resterilize.

Rx only

Pictorials are for reference only.

 B. Braun Medical Inc.  
 Bethlehem, PA 18018-3524 USA  
 1-800-227-2862

**FIGURA 2**

 Irving: 8200 Tristar Drive, Suite 120  
 Texas 75063

Irving,

 Fort Worth: 3515 NW Jim Wright Freeway  
 Fort Worth, Texas 76106

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